

Pediatric Aluminum Dual Pyramid Receiver Instructions

WHAT'S IN THE BOX

One of the following:

Pediatric Aluminum Dual Pyramid Receiver, 27 mm (PPF-138027)

Pediatric Aluminum Dual Pyramid Receiver, 51 mm (PPF-138051)

Instructions

Notice: WillowWood pediatric components have been tested and approved for use only with other WillowWood pediatric components. The use of WillowWood pediatric components with any manufacturer's adult-sized components will void the warranty. The use of WillowWood pediatric components with another manufacturer's pediatric components is done so at the sole judgment of the attending prosthetist.

INSTALLATION

Connect the Pediatric Dual Pyramid Receiver to the pyramid components of choice. Once alignment has been achieved, apply Loctite 242 (or equivalent) to all pyramid setscrews and tighten the setscrews to 3 ft-lbs (4 Nm).

DIFFERENT LENGTHS AVAILABLE

There are two available lengths of Pediatric Aluminum Dual Pyramid Receivers: 27 mm and 51 mm. If the length that was initially ordered is inappropriate, please contact WillowWood to order the other length.

WARRANTY

The warranty for the Pediatric Aluminum Dual Pyramid Receiver is one year from date of invoice. The use of WillowWood pediatric components with any manufacturer's adult-sized components will void the warranty. Use of the Pediatric Aluminum Dual Pyramid Receiver for amputees whose adjusted body weight is greater than 132 lbs (60 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendations and will void the one year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands



PN-1667-D 16 NOV 2012